

Fasmedo[®] **Medical Originality**

D5 Micro Camera



Quick Start Guide

See more information on www.fasmedo.com or scan



Fasmedo



Fasprosystem



Fasmedo



1. Package contents



1. D5
2. Bluetooth pedal
3. Video recording box(Optional)
4. Remote control for the video recording box(Optional)
5. Front protecting cover
6. Rear protecting cover
7. HDMI cable (white)
8. D5 charging & transmission cable (USB Type-C, white)
9. Video recording box power cable (mini-USB, black)
10. Bluetooth pedal charging cable (USB Type-C, black)

2. Appearance introduction



3. Before using your Camera


A. Pull to remove the sticker before charging or using the D5 for the first time.



B. Charge the battery fully at least 5 hours before using your D5 for the first time.

C. We suggest NOT to charge the battery with computer.

4. Replace the battery

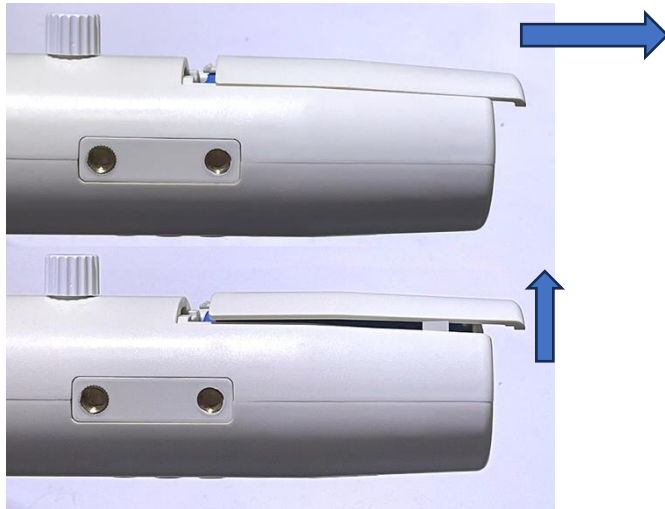

- A. Hold the battery door and slide right to release it first, then lift up to open the battery door.



- B. Remove the original battery and insert new one back carefully, note the “+” and “-” signs printed inside the slot, with the “+” side near the rear end of the camera. Close the battery door after that.

5. Turn on your D5



A. Working Modes: Working mode will be selected when power-ON. D5 can work in 2 different modes:

A-1. HDMI output mode:

Please first turn ON the monitor (or video recording box) and make sure HDMI cable is connected between D5 and the monitor (or video recording box).

A-2. Wi-Fi preview/control mode.

B. Turn ON/OFF the camera: Press and hold the Power button for 3 seconds to turn ON/OFF your D5. Once on, the LED should show a solid green light for Wi-Fi mode, or solid violet light for HDMI mode.

C. Turn OFF/ON Auto Focus: When image is in auto focus, you can quick-click the Auto Focus Switch Button (same as Power button) to turn OFF auto focus, and the camera will stay in that focus range.




When turn OFF the auto focus, the indicator (the left button of the switch button) will show White light to remind you that the auto focus status is OFF.




When object distance changed, you can just quick-click the switch button(same as Power button) again to turn ON auto focus.

D. Control the light: Push the light intensity tuner to turn ON/OFF the light, turning the knob clockwise to increase intensity

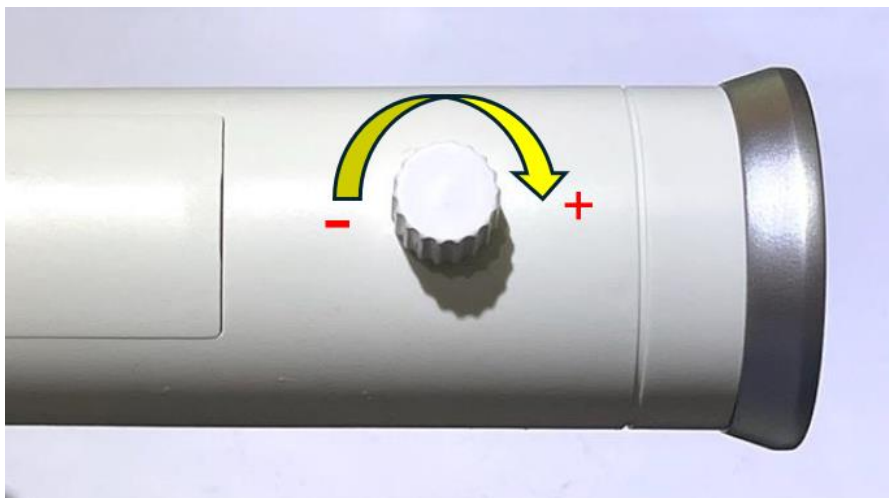

(beep twice when reached maximum), turning the knob counterclockwise to decrease the intensity (beep once when reached minimum).

E. Using the video recording box: Connect the video recording box to USB power source (5V/1A), connect its HDMI input port to your D5 and output port to TV/monitor respectively, and then plug-in an USB disk or pen drive for video recording:

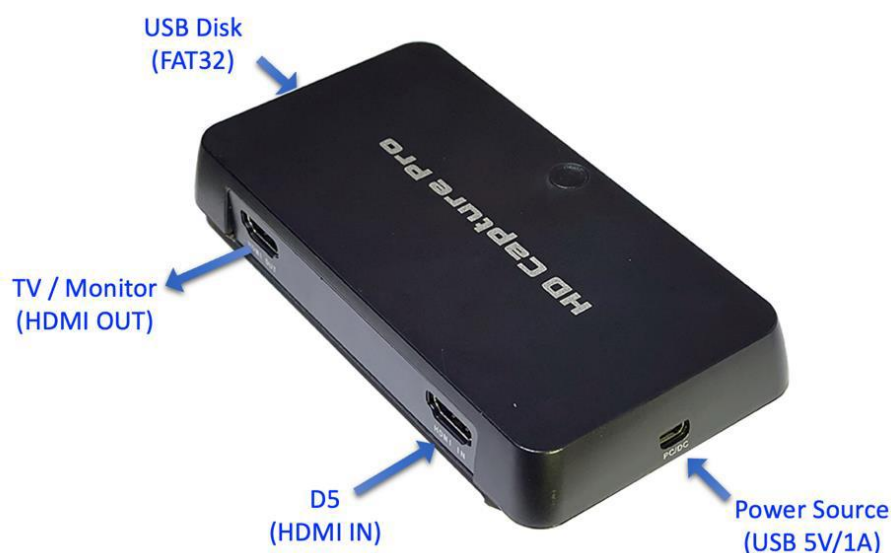

Turn ON your TV/monitor first, and then turn ON your D5, you should see video displayed on the TV / monitor within 10 seconds.

6. Connect your D5 with App


(Go to Step 7 if you have APP paired with Camera already.)

6.1 Download the App

- A. iOS : Search for “Fasmedo Cam” in the App Store to download the app to your smart device.
- B. Android : Search for “Fasmedo Cam Lite” in the App Store to download the app to your smart device.
- C. Windows 10 : Download Windows 10 APP from <https://www.fasmedo.com/apps/>.
- D. Mac : Download Mac APP from <https://www.fasmedo.com/apps/>.

6.2 Pair your Camera


The start screen should show a blank list.
Tap the  icon in the top right corner of your screen to add a Camera.

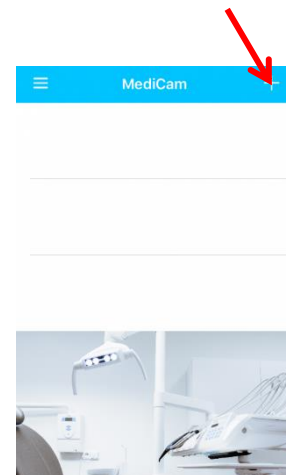


6.3 Search & Connect

Make sure Wi-Fi and Bluetooth are enabled on your smart device or PC.

- a. New Camera is found. Tap on it to continue.
- b. Confirm to “Join Camera’s network”.

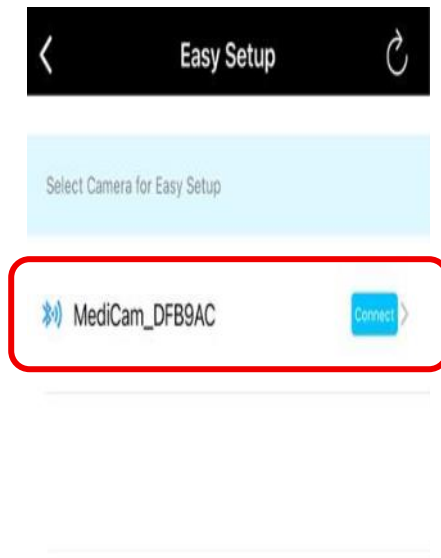

Now you see a Preview on your smart device or PC screen for the area being filmed.

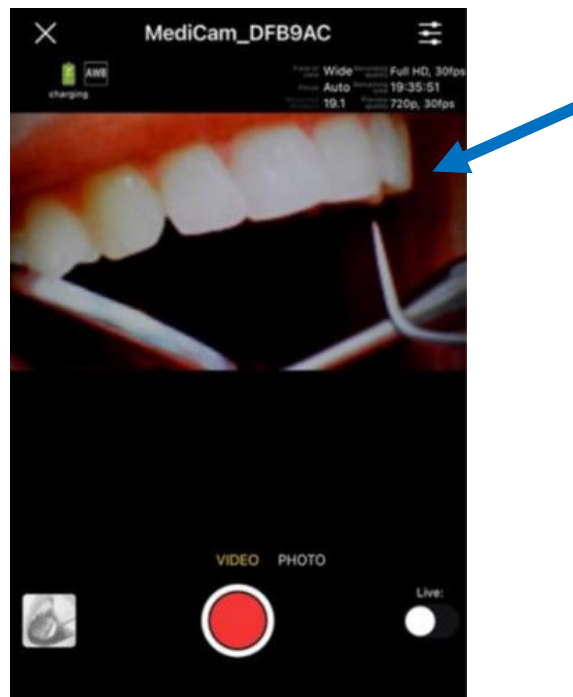

7. Operation your Camera

7.1 By clicking the Recording button

Press the Recording button to start recording, press it again to stop recording. (A high-pitched tone will sound to signal the start of recording; A low-pitched tone will sound to signal the stop of recording.)



7.2 By tapping the screen on the app

When in the preview mode, videos can be taken by tapping the red record button and photos can be taken by camera button.

7.3 By using the Pedal

A. Before using the Pedal, have it charged at least 3 hours.

B. Press the POWER switch to turn on the Pedal for set up.

C. Bluetooth Pedal Pairing Process:



Open the setting page



Click the Bluetooth open



Connect Name: **Bluetooth Music Pedal**

If you want to reconnect to other device, you need to press “reset” of pedal for 3 seconds and remove the original connect record.



- D. Stepping on the left pedal to start recording, and stepping again on the left pedal again to stop recording; stepping on the right pedal to take photos.

7.4 By using the Video Recording Box(Optional)

With the video recording box connected, you can push the button on the box to start/stop recording, or push “REC” button on the remote controller to start, and push “STOP” button to stop recording.



8. Device Settings

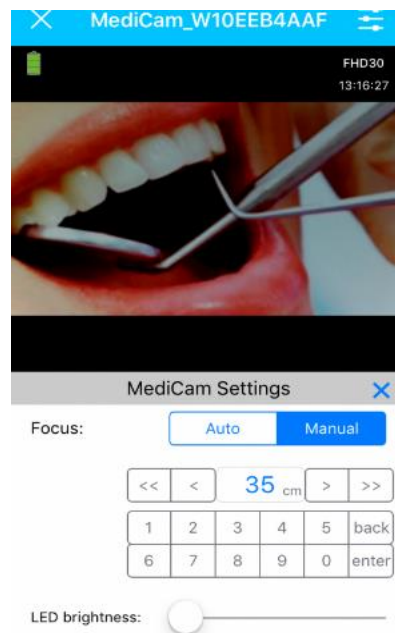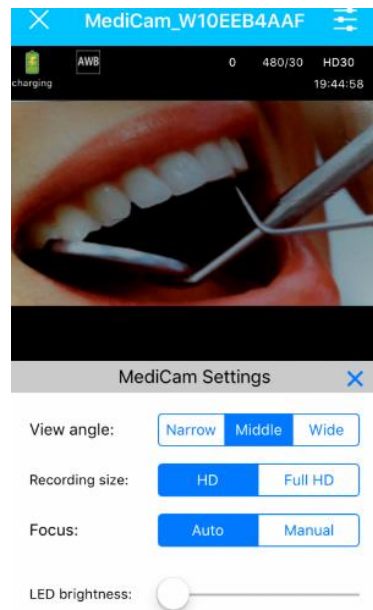

On the preview screen, tap the Tools icon() on the top right corner to configure the display of Videos/ Photos.


9. Manage Camera Videos/ Photos

9.1 Save media files from Camera to PC


- A. Connect your Camera to PC by USB cable.
- B. Turn on the Camera to access the media files stored in internal memory card.
- C. View the videos (VIDEOS folder) and photos (PHOTOS folder) on PC, select to view/save the files you need.

9.2 Save media files from app to your smart device

- A. On the Start screen, click the Gear icon  , tap “Manage Photo and Video” to save files needed to your smart device.

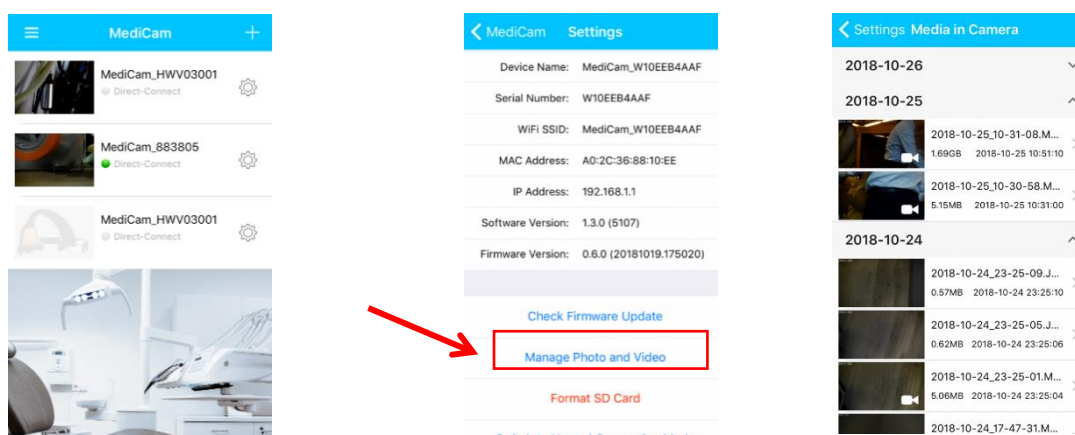

- B. Or in the Preview mode, tap the icon folder to save files needed to your smart device.

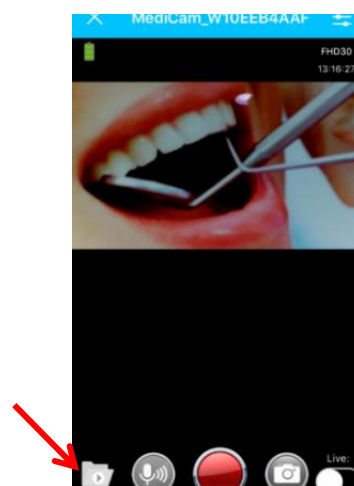

9.3 Delete the files from Camema

When the file is saved to a PC or smart device, remove the files in Camera to reserve space for recording new videos.

10. Live Streaming


Before start using Live Streaming, you must possess a cellular-4G/5G-enabled smart device and your smart device must be connected to a cellular 4G/5G network. (cellular plan NOT included)

10.1 Host Live Event



(Default view time is 1,000hours, each viewer would accumulate consumed time)

A. When viewing video streaming on app, tap “Live” to switch to live broadcasting.

B. The app will show “Prepare live streaming....” Then “Started live streaming...”


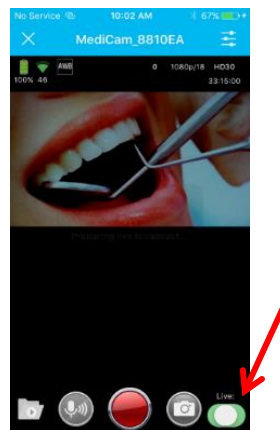


C. You will receive a Sharing code when live streaming starts. Tap on the “share” icon to share it with friends.

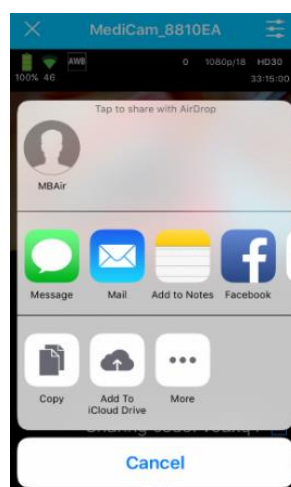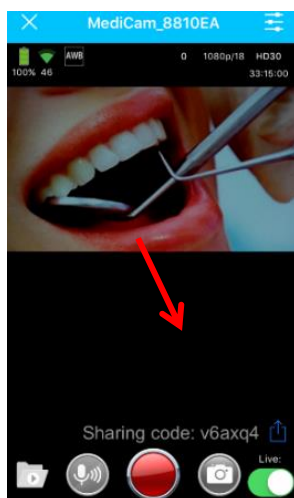


10.2 Watch Live Events

- A.** All invited parties must download Camera App before watching live events.
- B.** Tap “Live Events” to proceed, and tap the **+** icon to add live event, then enter the sharing code you have received.

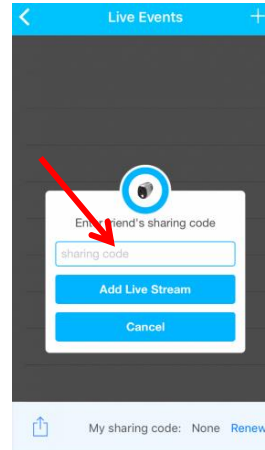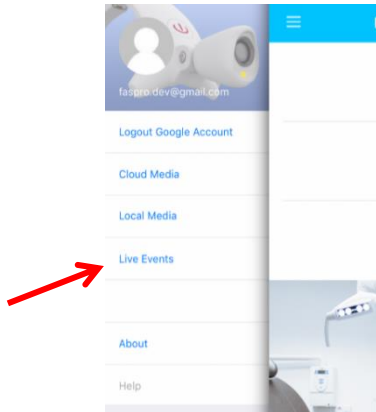

- C.** The Live event will be added to the list. The green light indicates the event is live now.
- D.** Tap on the live event will lead audiences to the live streaming view. There is no playback control. Tap the “X” icon to exit.

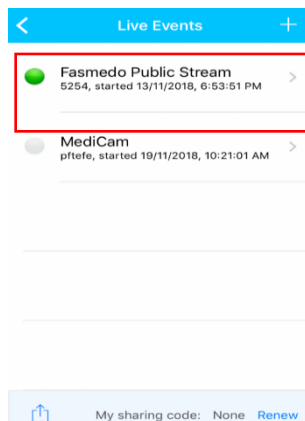

11. Button/LED Function

MODE	Power Button Indicator Behavior
Power ON (HDMI mode)	Boot up:  Ready: 
Power ON (Wi-Fi mode)	Boot up: Fast flash  Ready:  or  (battery<20%)
Recording	Battery>20 %: Slow flash  Battery<20 %: Fast flash 
Device OFF	Not charging:  Charging, battery <100 %: Slow flash  Charging, battery full: 
Memory Full	Slow flash 

12. Bluetooth Pedal LED indicator Function

LED Indicator	Status
Pairing Status	R-G-B flash sequentially 200ms loop
Connect Successful	The current battery color flash light 500ms, light off 1500ms loop
Reconnect	The current battery color flash light 600ms, light off 600ms loop
Disconnect	The current battery color flash light 200ms, light off 200ms loop

13. Safety Instruction

1. The wireless signal generated by the device may cause interference or malfunction of the medical device (such as a pacemaker or hearing aid), please use the Camera carefully.
2. This Camera device has been tested to meet the specific absorption rate (SAR) limits of body worn devices. The EU's maximum allowable level is 2.0 W/kg, and Camera is lower than this value
3. Not to look at LED directly at all times to avoid damaging your vision.
4. Avoid exposing Camera to fire and other direct heat sources, including hair dryers, microwave ovens and other appliances.
5. It is recommended that you do not disassemble the unit for any reason.
6. Do not force objects into your device connection, connection or button. If the connector is not easily inserted, it may not match.
7. Do not use spray cleaners on your Camera, as this could damage the system.
8. The LED of device complies with the requirements of Laser Class 1.

P/N: 500FMWD5003

All rights reserved©2018 Faspro Systems Co., Ltd.



TEL: +886-2-2517-0160

6F, No.160, Sec. 2, Nanjing E. Rd.,
Zhongshan Dist., Taipei, Taiwan
<https://www.fasprosystems.com>